


VENTURA 4 & VENTURA 4 DLX (S/N # VP108386 & Higher)

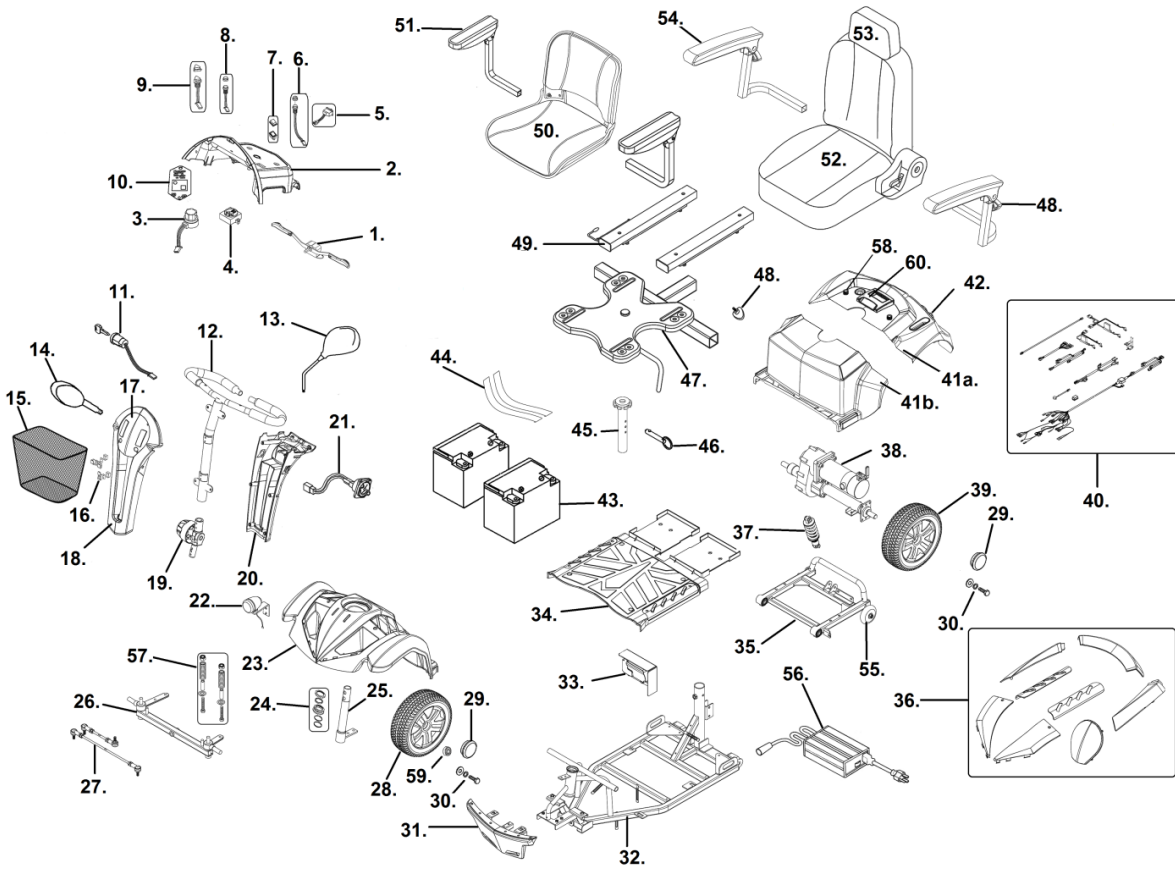

- | | |
|--|--|
| 1. Forward-Reverse Assembly- VENT-01
Throttle Lever Only- VENT-87 | 38. Motor-Transaxle-Brake Assembly- VENT-19N
Transaxle- C09-082-00200
Motor- C09-082-00500
Motor Brush- SF5012
Electro Magnetic Brake- C09-067-00400
Free Wheel Lever & Hardware- [4] |
| 2. Face Plate Assembly Folding Seat- VENT-04
Face Plate Assembly Capt. Seat- VENT-05 | 39. Rear Wheel Assembly Solid- VENT-28
Rear Wheel Assembly Pneumatic- VENT-57
Pneumatic Tire Only- VENT-59
Inner Tube Only- VENT-60 |
| 3. Speed Dial Assembly- S31001
Speed Dial Knob- LRM412103-11 | 40. Wire Harness Fish-On Seat- S18-082-00100
Wire Harness Capt. Seat- S18-082-00200 [1]
(includes tiller and battery harness)
Battery Harness Only- S41158B |
| 4. Battery Gauge LED- C11-076-00100
Battery Gauge Analog- C11-033-00101 | 41a. Rear Shroud Folding Seat- VENT-12N
Rear Shroud Capt. Seat- VENT-69N
Rear Shroud Decoration Rd Lt- [4]
Rear Shroud Decoration Rd Rt- VENT-91N [2]
Rear Shroud Decoration BI Lt- [4]
Rear Shroud Decoration BI Rt- [4] |
| 5. Turn Signal Switch Capt. Seat- VENT-72 | 41b. Battery Shroud- C08-082-01100
Bat. Shroud Decoration Rd Lt- C08-082-01218 [2]
Bat. Shroud Decoration Rd Rt- C08-082-01318 [2]
Bat. Shroud Decoration BI Lt- [4]
Bat. Shroud Decoration BI Rt- [4] |
| 6. Head Light Switch- S31122
Head light Switch Sub- GT8072W38
Headlight Switch Cover- GT8072W43 | 42. Rear light Capt. Seat- VENT-66
Rear Left Turn light Capt. Seat- VENT-67
Rear Right Turn Light Capt. Seat- VENT-68 |
| 7. Fast/Slow Switch- GT8072W41 | 43. Battery 35 AH (2 needed)- LRA402208 |
| 8. Horn Switch- S31012
Horn Switch Cover- GT8072W43 | 44. Battery Strap- VENT-09 |
| 9. Flasher Switch- GT8072W32 | 45. Seat Post- VENT-36 [3] |
| 10. Turn Signal Relay Capt. Seat- VENT-76 | 46. Seat Post Pin- S31152 |
| 11. Key Switch- S41160
Key- S4KEY | 47. Seat Plate Folding Seat- VENT-73
Seat Plate Capt. Seat- VENT-74 |
| 12. Handlebar Assembly- VENT-03 | 48. Arm Adjustment Knob- S31052 |
| 13. Mirror Right- VENT-53
Mirror Left- VENT-54 | 49. Seat Slides- VENT-65 |
| 14. Turn Signal Left- GT8072W35
Turn Signal Right- GT8072W36
Turn Signal Bulb- GT8072W53 | 50. Seat Assembly- VENT-41 |
| 15. Basket- S35003-PL | 51. Left Arm Assembly- VENT-29
Right Arm Assembly- VENT-30
Left Arm Pad- VENT-31
Right Arm Pad- VENT-32 |
| 16. Basket Mounting Bracket- VENT-55 | 52. Seat Assembly Capt. Seat 18"- VENT-42
Seat Assembly Capt. Seat 20"- VENT-43 |
| 17. Front Decoration Red- VENT-61
Front Decoration Blue- VENT-62 | 53. Headrest Captain Seat 18"- SEC2010N
Headrest Captain Seat 20"- [4] |
| 18. Front Tiller Cover- VENT-06 | 54. Left Arm Assembly Capt. Seat- VENT-33
Right Arm Assembly Capt. Seat- VENT-34
Arm Pad Capt. Seat- VENT-35 |
| 19. Tiller Swivel Assembly- VENT-02 | 55. Anti-Tip Wheels- VENT-16 |
| 20. Rear Tiller Cover- VENT-07 | 56. Charger- LRA402210 |
| 21. Charger Socket- S31147 | 57. Front Spring Set- VENT-77 |
| 22. Head Light Assembly- VENT-44 | 58. Shroud Knob- VENT-86 |
| 23. Front Shroud- VENT-10 | 59. Front Wheel Bearing- VENT-25 |
| 24. Stem Bearing Set- VENT-13 | 60. Frame Release Lever- C08-082-01400 |
| 25. Steering Axle Assembly- VENT-27 | |
| 26. Front Fork Assembly- VENT-23 | |
| 27. Steering Rod Assembly- VENT-26 | |
| 28. Front Wheel Assembly Solid- VENT-24
Front Wheel Assembly Pneumatic- VENT-56
Pneumatic Tire Only- VENT-59
Inner Tube Only- VENT-60 | |
| 29. Hub Cap- VENT-70 | |
| 30. Rear Axle Bolt- VENT-71 | |
| 31. Front Bumper Assembly- VENT-14 | |
| 32. Front Frame- VENT-47N | |
| 33. Controller- VENT-08 | |
| 34. Middle Shroud- VENT-11N | |
| 35. Rear Frame- VENT-18N | |
| 36. Shroud Kit Red- S08-082-00118
Shroud Kit Blue- [4] | |
| 37. Shock Absorber- VENT-15N | |

[1] Serial # VP058688 and higher.

[2] When stock is gone these parts will only be available with part #36.

[3] No longer available in chrome.

[4] Part is no longer available.