

# Buzzaround Extreme, Rental Bumper Kit

**Part Number: MBB-118R-KIT**

**Kit Includes:** 1- Front bumper, 1- Left-side bumper, 1- Right-side bumper, 1- Rear bumper, 4- Right-angle brackets, and all hardware.



16- Hex Head Bolts  
M8-1.25 x 20



16- Fender Washers



4- Set Screws  
M6-1.0 x 10



4- Hex Head Bolts  
M8-1.25 x 25



20- Split Locks

**Tools Required:** 13mm socket wrench, 13mm open end wrench, 6mm Allen wrench, 3mm Allen wrench


## To Install - Refer to figures 1 - 3.

1. Remove the two rear anti-tip wheels.
2. Remove the rear wheels. (Optional)
3. Align the four holes in the rear bumper and scooter frame. Insert four M8-1.25x25 bolts and four split locks from the outside of the frame, and tighten the four bolts to secure.




**Figure 1: Rear Bumper Installed**

4. Make sure the seat and battery packs are removed, then stand the scooter up as shown.
5. Place a right-angle bracket on the outside of the scooter frame, align the two holes in the bracket and frame. Insert two M8-1.25x20 bolts, split locks, and fender washers from the inside of the frame. *Do not fully tighten the bolts at this time.* Repeat step 4 for the other three right-angle brackets.
6. Place one side bumper on top of the right-angle brackets and align the four holes in the brackets and side bumper. Insert four M8-1.25x20 bolts, split locks, and fender washers from the underside of the bracket. *Do not fully tighten the bolts at this time.* Repeat step 5 for the opposite side.
7. Slide the front bumper onto the left and right side bumpers, push flush, and tighten the four set screws to secure the front bumper.
8. Fully tighten the sixteen M8- 1.25x20 bolts to secure the front and side bumpers to the scooter.

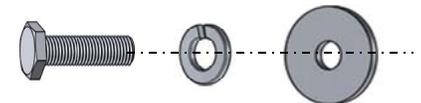

**Figure 2: Hardware Layout**

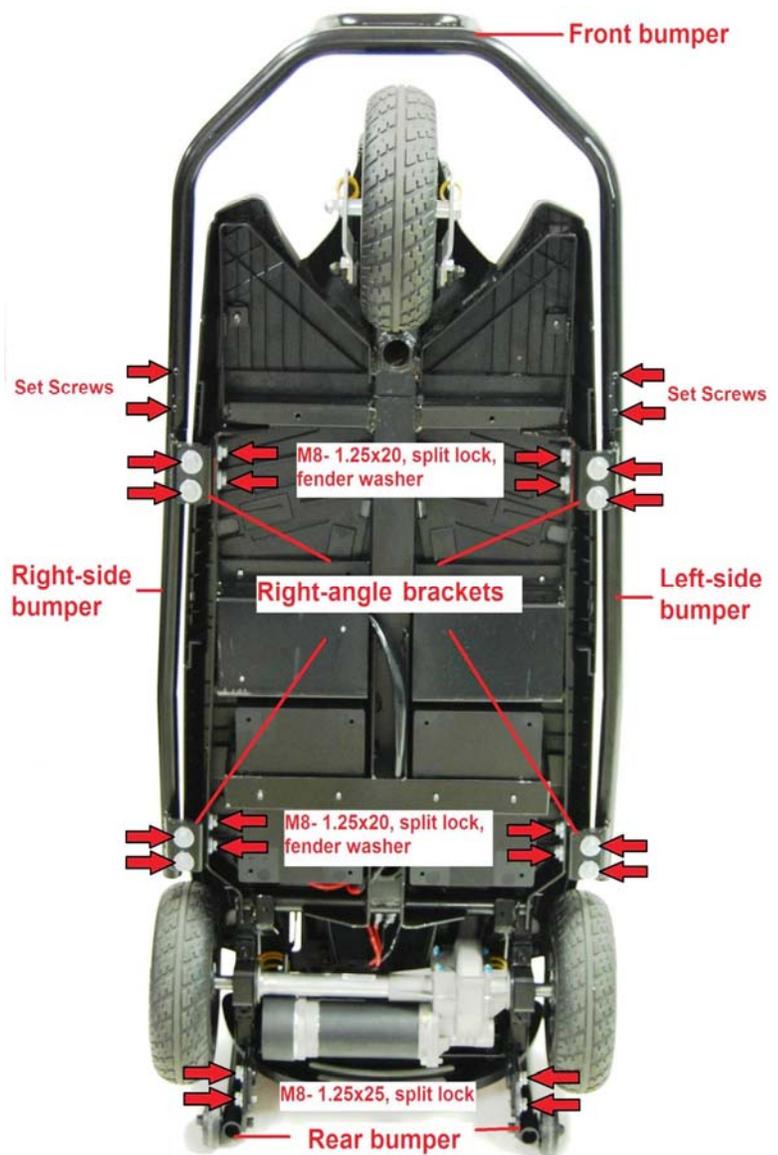

**Figure 3: Underside of Frame**

